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Ingrediënten :

ADDAX
LABORATOIRE

MYCOFLASH® ATHLETE
FOOT SPRAY
Medical device - Class IIA

- **Treats and protects**
- **Instantly reduces itching and neutralises unpleasant odours**
- **Hygienic and effective**

General advice: Please read this leaflet carefully before using MYCOFLASH® Athlete's Foot Spray, even if you have previously used this medical device. The information contained in a new package may have been revised to reflect the latest scientific knowledge and data. However, you must use MYCOFLASH® Athlete's Foot Spray correctly to achieve good results. Please keep this leaflet for future reference.

Application : MYCOFLASH® Athlete's Foot Spray is a medical device for the treatment of epidermophytes. Epidermophytia is generally associated with itching, redness and exfoliation between the toes and on the sole of the foot. As well as being contagious, epidermophytitis is also accompanied by unpleasant odours. Fungi spread easily in warm, damp places, such as swimming pools, showers and damp floors. MYCOFLASH® Athlete's Foot Spray reduces itching and neutralises unpleasant odours.

Mode of action: MYCOFLASH® Athlete's Foot Spray is based on natural ingredients or ingredients of natural origin, which help in the treatment of epidermophytes. MYCOFLASH® Athlete's Foot Spray forms a protective layer on the skin of the foot. The combination of ingredients penetrates the upper, calloused layer of the skin and creates an environment hostile to the development of the fungi that cause epidermophytia. NATILACT is a natural active ingredient that protects and moisturises the skin. The protective layer adheres to the skin, which binds the active substances and enables them to be effective.

Dosage and use :

Dry feet thoroughly and spray twice a day (morning and evening) MYCOFLASH® Athlete's Foot Spray Treatment in a thin layer on both feet. This is necessary because the fungus may also be present on the other foot without being visible. Spray correctly between the toes, on the sole of the foot and on the sides. Leave to dry for one or two minutes to form a protective layer.

Duration of treatment: Improvement generally occurs after a few days. Continue treatment until the epidermophyte has completely disappeared (generally after one week). Irregular use or premature cessation of treatment increases the risk of recurrence.

Use during pregnancy and/or lactation: No information is available on the use of MYCOFLASH® Athlete's Foot Spray during pregnancy or lactation. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using this product.

Alcohol Denat., Aqua, Glycerin, Urea, Natilact®, Phenoxyethanol, Ethyl-hexylglycerin, Camphor, Menthol, Lactic acid, 4-Terpineol.

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CE 0481

Side-effects : Possible side-effects include slight skin irritation and a temporary sensation of burning, stinging or itching.

⚠ Warnings:

- Keep out of reach of young children;
- For external use only;
- Do not use in the event of hypersensitivity or allergy to any of the ingredients; in the event of allergy, stop using the product immediately;
- Avoid contact with eyes and mucous membranes;
- Consult your doctor if there is no improvement after two weeks' use;
- Highly flammable liquid and vapours
- Keep away from heat, hot surfaces, sparks, naked flames and all other sources of ignition. Do not smoke;
- Avoid breathing mists / vapours / aerosols;
- Causes severe eye irritation;
- IF IN EYES: rinse cautiously with water for several minutes. Remove contact lenses. Continue rinsing;
- In the event of irritation or skin rash: consult a doctor.

Ingredients:

Alcohol Denat., Water, Glycerin, Urea, Natilact®, Phenoxyethanol, Ethyl-hexylglycerin, Camphor, Menthol, Lactic acid, 4-Terpineol.

Content: 25 ml

Medical device: MYCOFLASH® Spray for athlete's foot is a class IIA medical device in accordance with European Directive 93/42/EC concerning medical devices.

⚠ Storage instructions: store at room temperature (5-25°C), in a dry place protected from light. Keep out of reach of young children.

📅 See the expiry date on the bottom of the pack.

If you have any specific questions about this medical device, please call the information line: 0800-73183. This information number can be reached during office hours.

Date of last revision of this leaflet: November 2024 - V.11

📍 PK BENELUX BV Vluchtoord 17, 5406 XP Uden, NL

DISTRIBUTOR : ADDAX Laboratoire - 24 rue Erlanger - 75016 Paris - FRANCE
www.laboratoire-addax.com

CE 0481