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Ingrediënten:

Aqua, Ethoxydiglycol, ISK1320®, Pentylene Glycol, Alcohol Denat., tridiceth-9, PEG-40, Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, Natilact,

Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cabin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

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MYCOFLASH®
TREATMENT LIQUID
Medical device - Class IIA

LIQUID TREATMENT FOR NAIL FUNGUS

• **Proven effective against fungal infections responsible for nail fungus.**

• **A unique formula containing ISK1320®, also known as 'INSTANT KILLER'. This active ingredient penetrates the entire nail surface, killing fungi in just 20 seconds.**

• **Prevents contamination of other nails and protects against the reappearance of fungi. Contains essential oils.**

General advice: Please read this leaflet carefully before using MYCOFLASH® Nail Fungus Treatment Liquid, even if you have previously used this medical device. The information contained in a new pack may have been revised to reflect the latest scientific knowledge and data. However, you must use MYCOFLASH® Nail Fungus Treatment Liquid correctly to achieve good results. Please keep this leaflet for future reference.

Application:

MYCOFLASH® Nail Fungus Treatment Liquid is a medical device that assists in the treatment of nail fungus (onychomycosis). MYCOFLASH® TREATMENT LIQUID is effective in the treatment of fungal infections that cause nail fungus. The unique formula contains ISK1320®, also known as 'INSTANT KILLER', which kills fungi in less than 20 seconds. The liquid forms a layer on the nail, creating an unfavourable environment for fungi to grow in the nail plate and acts as a physical barrier to prevent the spread of fungi. MYCOFLASH® is easy to use, simply apply the varnish 2 or 3 times a day to the nail using the applicator brush. For best results, we recommend that you carefully file the infected nail once a week. Once the treatment has been stopped, the fungal infection can return very quickly, which is why it is important to continue using MYCOFLASH® Treatment Varnish until the infection has completely disappeared.

As damp environments are one of the causes of fungal nail infections, it is strongly recommended that you wear socks made only of absorbent materials, such as cotton or wool. If you have a fungal nail infection, it's important to keep your feet dry.

Dosage and use (for adults only):

MYCOFLASH® liquid treatment is for external use only. Apply the product to the affected area two or three times a day (e.g. morning and evening) until the mycotic nail has completely grown out.

Thanks to the ISK1320® formula, which kills fungus instantly in 20 seconds, the applicator brush does not become infected, preventing infection/re-infection of other nails and protecting against any reappearance of fungus. MYCOFLASH® nail fungus treatment liquid can be combined with oral antifungals.

Mode of action: Thanks to a unique combination of ingredients, MYCOFLASH® Treatment Varnish forms a layer which creates an environment hostile to the development of fungi in the nail plate. It also forms a physical barrier that inhibits the spread of fungi. MYCOFLASH® Treatment Varnish reduces the acidity of the nail, creating an unfavourable environment that stops the growth and spread of fungi.

Use during pregnancy and/or lactation : No information is available on the use of MYCOFLASH® Treatment Varnish during pregnancy or while breastfeeding. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using this product.

Side effects: Possible side effects include erythema, irritation, oedema, pain, nail discolouration and onycholysis.

Do not use Nail Fungus Treatment Liquid if you are allergic to one or more of the ingredients.

⚠ Warning:

- For external use only;
- Do not use if you are hypersensitive or allergic to any of the ingredients;
- Keep out of reach of young children;
- Avoid contact with eyes and mucous membranes;
- Do not use on open wounds or damaged skin;
- Do not bandage tightly;
- Avoid applying heat to the application area;
- Consult your doctor if there is no improvement after two weeks' use.

Ingredients :

Water, Ethoxydiglycol, ISK1320®, Pentylene Glycol, Alcohol Denat., tridiceth-9, PEG-40, Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, Natilact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cabin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

Content : 5 ml

Medical device: MYCOFLASH® Treatment Varnish is a class IIA medical device, in accordance with European Directive 93/42/EC relating to medical devices.

Storage instructions: store at room temperature (5-25°C), in a dry place protected from light. Keep out of reach of young children.

⏰ See the expiry date on the bottom of the pack.

If you have any specific questions about this medical device, please call the information line: 0800-73183. This information number can be reached during office hours.
Date of last revision of this leaflet: November 2024 - V1.2

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